

PRISCA 5.2 SP2 and 5.3.1 on Microsoft Windows 11

In an ongoing effort to keep customers informed, Siemens Healthineers is providing this information regarding the PRISCA 5 prenatal risk calculation system software. PRISCA 5 software versions 5.2 SP2 and 5.3.1 are supported on Microsoft Windows 11 operating systems, including Windows 11 Enterprise and Pro Editions.

Ensure all operators review and understand the information presented in this document. This document is a supplement to the operator's guide. Place this document with the PRISCA 5 Operator's Guide and inform everyone in your laboratory of this guide. If you have any questions, contact your local technical support provider or distributor.

If you have any questions or need additional information, please contact your local technical support provider or distributor.

www.siemens.com/document-library

Printing Patient Reports as PDF in Simplified Chinese

NOTE: The instructions provided are only for PRISCA 5 systems running software version 5.3.1 on Windows 11.

NOTE: The instructions provided are only for Simplified Chinese.

Siemens Healthineers determined that some PRISCA systems running software version 5.3.1 on Windows 11 may incorrectly generate patient reports in Simplified Chinese. The reports may contain question marks ("?") in place of Simplified Chinese text.

On performing installation with the language set to Simplified Chinese and experiencing the incorrect generation of patient reports, ensure correct installation of Simplified Chinese (Page 1 *Installing Simplified Chinese on the PRISCA System*) and, if the incorrect production of reports persists, reconfigure patient reports (Page 2 *Printing Patient Reports as PDF in Simplified Chinese*).

Installing Simplified Chinese on the PRISCA System

1. Install Prisca 5.3.1.5 Custom (Chinese).
2. In Windows 11, navigate to **Control Panel > Clock and Region > Region > Administrative**.
3. Select **Change system locale...**
4. Select **Chinese (Simplified, China)** and select **OK**.
5. Restart the Windows 11 system.
6. Once the Windows 11 system has restarted, launch the PRISCA application.

7. Navigate to **Settings > Configuration > Report Settings**.
8. Under Print PDF to file, select **Yes**.
9. Ensure Screen Preview is set to **No**. Ensure Show Printer Dialog is set to **Yes**. Ensure Archive reports is set to **Yes**.
10. Select **OK > Yes**.
11. Navigate to **Settings > Configuration > System Configuration > Language**. Select **CHT**. Select **OK**.
12. Restart PRISCA.
13. Create a new patient record.
14. Select **Report**
15. Select **Open File**. Select **OK**. If problems persist, see Printing Patient Report in Simplified Chinese.

Printing Patient Reports as PDF in Simplified Chinese

1. In the Patient window, create a new patient record and select **Save**.
2. Select **Report**.
3. Deselect the check box for **Print to File**. Advanced Options will display.
4. Select **Print to File**. Ensure **Type** is set to PDF File.
5. Select the check box for **Open File**.
6. Select **Scale Images** and **Unicode Text**.
7. Select **OK**. The PDF report now displays all appropriate fields in Simplified Chinese.

Trademark Information

All trademarks and brands are the property of their respective owners.