

# MAGNETOM Sempra, 1.5 Tesla

**Basic Planning Information** 



Table of contents	
Planning Example	3
Room Dimensioning	6
Statics and Transport	6
Cooling Water Installation	8
Air-conditioning	9
Electrical Installation	10
Noise Emission Values	10
Fringe Field	11
Siting Requirements	12
RF-Shielding	14
Quench Pipe	15
General Information	15

			Legend		
	Motion area / Swivelling ra	ange / Minima	al room size / Safety dis	tance	
	Service area		Ceiling mounted		New construction
	Additional equipment		Wall mounted		Demolition
			Floor mounted		
		Di	mensioning		
All installat	ion measurements apply to	finished wall	/floor/ceiling and are to	be checked prior	to assembling the unit.
Orientation point = reference point of the Siemens Healthineers unit for planning and installation					
Please not	e: The drawing parts in this	document ar	e not to scale!		













MAGNETOM Sempra - Equipment Legend						
	Weight (kg), Heat dissipation to the air (W)					
Pos.	Description	kg	W	Remark		
1.01	Magnet	4380	3000	#1/#2		
1.02	Fixed patient table Partial Drive	290				
1.03	RF-cabin					
1.04	RF-door					
1.05	RF-window					
1.06	RF-System filter plate	130	250	#7		
1.07	Magnet stop					
1.08	Electronics cabinet ECA	750	1500	#1		
1.09	ICS-passive cabinet	280	1500	#2/#4		
1.10	Power distributor	52		by customer		
1.11	Control unit MR AWP	20	200			
1.12	Host PC MR AWP	22	700	max. magnetic field strength 1mT		
1.13	Intercom system					
1.14	Alarmbox	1				
1.15	Air conditioning cabinet			by customer		
	<ul> <li>#1 Heat dissipation depending on measuring</li> <li>#2 Additional water cooling system necessary</li> <li>#4 The type of the cooling must be cleared with the Siemens project manager.</li> <li>#7 Installation of non-SIEMENS components prohibited</li> </ul>					



## Room Dimensioning

#### Technical minimum finished room height

Examination room min. 240 cm, Control room min. 210, Equipment room min. 220 cm.

#### Technical minimum finished room size

Partial Drive, Fixed Patient Table: 595.1 cm x 353.5 cm

## Statics and Transport

St	atics not to scale		
Support feet	and floor load		
Transport weight of the magnet: ~ 4600 kg	↓ 55.5 ↓ 55.5 ↓ ↓ ↓ Service end		
Installation weight of the magnet: 4380 kg 4 support feet each 15 cm x 25 cm (375 cm <sup>2</sup> ) Pos. 1 = 1157 kg Pos. 2 = 1025 kg Pos. 3 = 1033 kg Pos. 4 = 1165 kg	$\begin{array}{c ccccccccccccccccccccccccccccccccccc$		
You have to consider the additional weights of the RF cabin and the possible iron shielding for the static			

It is only possible to position the magnet on Stop-Chocs if it is installed in a non-magnetic RF cabin and if there is no iron shielding below the magnet.

Standard configuration is an installation on Sylomer/Sylodamp.

#### **Building vibrations**

External vibrations or shocks affecting the magnet may degrade image quality. In the three spatial orientations the building vibration must not exceed the following specification: Building vibration specification:

a max = -80 dB(g) in the frequency range from 0 to 100 Hz

The requirement for a  $_{max}$  is depending on the frequency.







# Cooling Water Installation

Cooling water (ICS Passive)				
A ICS Active cabinet and a External Fan-Unit is required if cooling water cannot be supplied by local cooling circuit for the ICS Passive cabinet				
Water quality Primary water	pH-value Hardness Chlorine portion Filtration Water / antifreeze	: 6 bis 8 : < 250 ppm CaCO ,< 14 °dH : < 200 ppm : 500 μm : 40% ethylene glycol 60% water		
Water quality Secondary water	Water to be used Filtration Water / antifreeze Additive for secondary chilled water circuit	: De-ionized water : 500 µm : n.a. : NaHCO <sub>3</sub>		
ICS Passive	Heat dissipation to water Water flow rate Water supply temperature Primary water pressure Pressure loss across ICS Pass Water temperature gradient	: 22,15 kW : 30 l / min : 6 to 12 °C : max. 6 bar sive : < 1 bar (typical 0.3 bar) : n.a.		

Cooling water (ICS Active)					
A ICS Active cabinet and a External Fan-Unit is required if cooling water cannot be supplied by local cooling circuit for the ICS Passive cabinet					
Water quality       Water to be used       : De-ionized water         Filtration       : 500 µm         Antifreeze concentration       : 40 % ethylene clycol 60% water					
ICS Active	Heat dissipation to water Water flow rate Water supply temperature Primary water pressure Water temperature gradient	: 22,15 kW : not manipulable : not manipulable : max. 6 bar : not manipulable			





# Air-conditioning

Environmental conditions					
In the MR-area must be ensure follow	ving conditions (system during operation)				
Examination room	Room temperature18 to 22 °CRelative humidity40 to 60 %Absolute humidity< 11.5 g/kg				
Technique room	Room temperature15 to 30 °CRelative humidity40 to 80 %Absolute humidity< 11.5 g/kg				
Control / Evaluation room	Room temperature15 to 30 °CRelative humidity40 to 80 %Absolute humidity< 11.5 g/kg				
The operating values should be set w	ithin these limits and ventilation must conform to local standards and regulations.				
Air filtering	In the equipment area: filter classification EU 4 (DIN 24185 / part 2) to filter out dust particles > 10 $\mu$ m. For the MR examination room observe the local regulations.				
Typical heat dissipation of the MR-components to the environment during an operation.	Examination room3 kWControl room2 kWTechnique room3 kW				
environment during an operation.       Technique room       3 kW         40       rH = 10 % 20 % 30 % 40 % 50 % 60 % 70 % 80 % 90 % 100 %         35					
0 2 4 0 0 10 12 14	x <sub>S</sub> / [g/kg]				



# **Electrical Installation**

		Power requirements		
Mains: TN-	S	3/N/PE AC 50/60 Hz ± 1 Hz	Connection value:	
Line voltage:		400 V ± 10 %	passive: Power consumption	29 kVA
Line to line	unbalanced:	max. 2 %	for time up to < 3 s:	29 kVA
passive	Line impedance:	≤ 250 mΩ	active:	39 kVA
active	Line impedance:	≤ 200 mΩ	Power consumption for time up to < 3 s:	43 kVA
Only coppe	r cables are allowed.			
The size of the terminals in the EPC are designed for 35 mm <sup>2</sup> .		Measurement sequences	< 3 s.	

Cable cross section is to be determined by national regulation and calculation.

#### Room lighting

Ambient lighting in rooms with diagnostics or with workstations must comply with the respective local and national regulations.

General requirements like the needed intensity of illumination - adjustable, reproducible, flicker-free or a limitation of dazzlings and reflections etc. have to be observed (EN 12464-1, DIN 5035-7).

# Noise Emission Values

Noise emission values					
If required, noise reduction should be realized based on the noise emission values as specified.					
Average values Examination room Control room Technique room					
across 8 hours	≤ 83.9 dB(A)	≤ 55 dB(A)	≤ 65 dB(A)		



# Fringe Field

## Requirement for magn. field level warning signs in the control zone ≥ 0.5 mT/ 0.9 mT

Limit for persons with cardiac pacemaker or insulin pump.

If the magnetic flux density in a given area exceeds 0.5 mT or 0.9 mT, it is necessary to display warning signs and restrict access in accordance with local regulations.

Fringe field distribution MAGNETOM Sempra				
Fringe field	Distance [m] from magnetic center in direction of			
	radial (X/Y-axis) axial (Z-axis)			
40mT	1.30	1.70		
20mT	1.40	1.90		
10mT	1.50	2.20		
5mT	1.70	2.50		
3mT	1.80	2.80		
0.9mT	2.25	3.55		
0.5mT	2.50	4.00		
0.05mT	4.10	7.00		



## Siting Requirements

#### Siting requirements for the magnet

The siting of the magnet must be such that during operation neither external influences affect the homogenity of the magnetic field nor the safety of persons and/or the functioning of sensitive equipment can be affected by the stray magnetic field.

#### Disturbing influences on the magnetic field

#### Static

E.g. steel beams, reinforcements, especially beneath the magnet. Partially correctable by shimming of the magnet and/or compliance with minimum clearances/maximum weights.

Dynamic

E.g. moving ferromagnetic objects, electrical wiring, transformers. Avoidable when minimum clearances are observed. Minimum distance depend on moving direction and magnet orientation. If distances are not kept please contact Siemens Healthineers, Planning Department.

		Minimum	clearance	
	Object	radial (X/Y)	axial (Z)	Max. weight
	Water cooling system	4.0 m	4.0 m	
Guidelines	Wheelchairs, beds, angiography systems	4.9 m	5.8 m	
for minimum	Carts up to approx. 200 kg	5.3 m	6.5 m	
clearances	Transformers < 1600 kVA	5.0 m	5.0 m	
ano maximum	AC cables < 1000A	2.5 m	2.5 m	
weights	Cars up to approx. 900 kg, CT	5.5 m	7.5 m	
	Trucks up to approx. 4500 kg, Lifts	6.2 m	9.0 m	
	Cyclotron	20.0 m	20.0 m	
	Street cars, trains	40.0 m	40.0 m	#1
	Angiography systems with magnetic navigation	30.0 m	30.0 m	
	Reinforcement distributed in thickness of floor slab	> 1.25 m be magnet ce	elow #2 nter	≤ 100 kg/m²
	Steel beam	> 1.25 m b magnet ce	elow #2 nter	≤ 100 kg/m

#1 The DC disturbances must not exceed a peak-peak value of 1250 nT (axial) and 2500 nT (radial). Occasionally these values might be exceeded although the minimum distances to DC sources are kept as stated in the planning guide. Please contact the planning department of Siemens Healthineers if the distances to trains, tramways or subways are smaller than 100 m.

#2 This minimum distance is required for shimming.Distance for magnetic shielding has to be adjusted according to individual shielding requirements.



Minimum distances magnet - magnet (Siemens Healthineers)							
	0.2 T	0.2 T 0.35 T 1.0 T 1.5 T 3.0 T					
0.2 T	10 m	10 m	5 m	6 m	10 m		
0.35 T	10 m	10 m	5 m	6 m	10 m		
1.0 T	5 m	5 m	4.5 m	5 m	6 m		
1.5 T	6 m	6 m	5 m	5 m	6 m		
3.0 T	10 m	10 m	6 m	6 m	6 m		
7.0 T	10 m						

No magnet is ramping during the other runs applications! Shim is only optimized with both magnets ramped up during the shimming procedure.

Guidelines for max.Permissible Magnetic Flux Density (mT)			
mT	radial (X/Y)	axial (Z)	
40	1.30 m	1.70 m	Servoventilator
20	1.40 m	1.90 m	Defibrillator
10	1.50 m	2.20 m	RF-filter plate
5	1.70 m	2.50 m	MR electronics cabinet (Siemens Healthineers) GPA/EPC, SEP
3	1.80 m	2.80 m	Small motors, watches, cameras
0.9	2.25 m	3.55 m	Pacemakers and insulin pumps, limit for public access
0.5	2.50 m	4.00 m	Pacemakers and insulin pumps, X-ray tubes, limit for public access
0.05	4.10 m	7.00 m	Gamma cameras, linear accelerators

The magnetic stray field is present in all three dimensions around the magnet and can be reduced by a magnetic shielding. Typical lines of constant magnetic flux density are shown in the drawing. This represents the ideal field distribution in air, which can be distorted by the presence of steel in the building. Magnetic field specification depends on manufacturer.

#### Disturbances caused by the stray magnetic field

All equipment and systems whose functions could be influenced by external magnetic field must be taken into consideration. The maximum permissible magnetic flux density depends on the sensitivity of each system component and must be clarified if necessary with equipment manufacturer.

#### Site inspection

In critical cases the site must be inspected on customer's expense by Siemens Healthineers or one of Siemens Healthineers appointed representatives to ensure basic suitability on the site.

This inspection is exclusively concerned with the measurement of the magnetic and radio frequency interference and building vibrations.

This inspection of other construction requirements, in particular the static and air conditioning and also the performance and supervision of on-site installation preparations and the later compliance with the basic operating requirements is not our responsibility.



## RF-Shielding

#### **RF-shielding**

An RF-shielding (faraday cage) is required for the MR-examination room. This shielding protects the environment from RF interference and conversely protects the MR system from external interference.

Required attenuation: >90 dB over the frequency range 15 to 128 MHz (>100 dB at Co-Siting).

These values must be certified by measuring before the MR system is installed.

RF-shielding components (doors, windows, interfaces) and complete modular RF-cabins can be supplied on request by Siemens Healthineers.

#### RF-Door

All RF-doors leading into the examination room have to be equipped with a door switch for indicating the closed / open position of the RF-door! It must be possible to lock the RF-doors from the outside. In addition to that it must always be possible to open the RF-door without key or additional devices in any cases from the inside!

The opening direction of the RF-door has to be to the outside of the RF-room.

Doors that open inwards is a safety risk due to room overpressure. For these rooms a pressure relief panel 600 mm x 600 mm (minimum size) must be installed into the RF cabin.

The RF-door is an important component for a good image quality and also for safety aspects. The customer/user of the MR system has to be informed to maintain the maintenance intervals given by the RF-room enclosure manufacturer. This will guarantee a correct function of the RF-door.

Flooring

An antistatic floor covering is necessary. The floor in the vicinity of the magnet and patient table (3.0 m x 5.8 m) must be levelled to within max.  $\pm 2 \text{ mm}$ .

The loading capacity of the flooring must be designed with the weight of the respective system components in mind.





## Quench Pipe

#### Quench pipe

A thermally insulated tube (quench pipe) made of non-magnetic metal must be fitted from the super-conducting magnet to the outside of the building in order to vent the vaporizing helium gas. Exact design information must be obtained from Siemens Healthineers Project Manager.

Cryogens

Liquid helium (He) and also helium gas are required for operation of the superconducting magnet. The transport of these liquid gases to the examination room requires the use of special vessels. The size and weight of the vessels should be checked with the local cryogen supplier.

If the magnet can not be filled from the left service side, a long helium transfer line has to be ordered for the refilling process!

## **General Information**

#### Display screen workstations

For setting up display screen workstations, take account of the guidelines in the Display Screen Workstation directive as well as any national regulations (e.g. EN ISO 9241-5).

#### Smart Remote Services (SRS)

Smart Remote Services (SRS) is used for remote diagnostics as well as remote service to provide highest system availability.

Requirements:

- Broadband connection (minimum 4 MBit/s down- and 768 kBit/s upstream, optimum 30 MBit/s down- and

2 MBit/s upstream) without time or volume limitations

- Router (for exclusive use with SRS)

Data protection and security is defined in the Smart Remote Services security concept.

## Network Integration

The Siemens Healthineers components are using TCP/IP Protocol, a 100/1000 Mbit/s switched Ethernet network and static IP addresses.

The required network cabling (min. CAT 5 TP) has to be provided on site. Media converters, which are needed for using fibre optic cabling, are not in scope of delivery.

To prepare the implementation of the new system into the existing network environment, the availability of the needed network data at least two weeks before starting the installation is mandatory.

This is the only way to ensure a seamless integration of the new system into the workflow of the department.

## Safety distances

Distances from moving parts of the medical device to walls, furniture and other equipment have to be kept to avoid injuries by crushing in compliance with local regulations, e.g. a minimum distance of 50 cm according to DIN EN ISO 13854.

It is the customer's responsibility to ensure the above requirements are followed. This is to avoid the risk of injury.



If safety distances are not maintained appropriate on-site safety measures have to be put in place. Clear visible markings according to national guidelines, e.g. crushing warning signs, hazard warning tape, hazard area cordon, safety mats, may be required.



#### Site readiness guidelines

The following general conditions are necessary to have the status of "Ready site":

- 1) Proper power available at Siemens Healthineers Equipment Power Cabinet location and all power outlets functioning 2) Air conditioning / humidification systems complete, tested and functioning properly according to Siemens
- Healthineers specifications.
- 3) RF enclosure, infrastructure of the examination room complete.
- 4) The quench line must be available for immediate use to allow suitable venting for the magnet during installation.
- 5) Plumbing complete except for any final connections to Siemens Healthineers equipment.
- 6) All cable trays, ducts, conduits correctly sized, located and installed according to the Siemens Healthineers drawings7) Room for equipment installation and immediate vicinity is dust-free and is to remain so for the duration of the
- installation.8) Customer approval for Siemens Healthineers Remote Service (SRS) connection and customers IT. Contact information and IP adress established.

#### Notes on preparations for installation

Contracts for performing and supervising on-site installation preparations should be concluded with technically competent companies by the customer. The customer is responsible for timely and proper completion and supervision of all preparations for installation at the construction site in observance of all applicable legal regulations (e.g. X-ray regulations, radiation protection regulations) and all applicable general recognized rules of technology (e.g. VDE regulations, DIN standards).

Execution and supervision of installation preparations at the construction site and later observance of the standard operating conditions are not included in our duties. The customer is responsible for checking the static calculations and, where applicable, the air conditioning in the building to be equipped.



On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice. Some/all of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features which do not always have to be present in individual cases.

The statements by Siemens' customers described herein are based on results that were achieved in the customer's unique setting. Since there is no "typical" hospital and many variables exist (e.g, hospital size, case mix, level of IT adoption), there can be no guarantee that other customers will achieve the same results. The customers cited are employed by an institution that might provide Siemens product reference services, R&D collaboration or other relationship for compensation pursuant to a written agreement.

Siemens reserves the right to modify the design, packaging, specifications, and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Not all features shown in this brochure are necessarily standard and available in all countries.

**Published by** Siemens Healthineers AG SHS ES FD

Siemens Healthineers Headquarters Siemens Healthineers AG Siemensstr. 3 91301 Forchheim, Germany Phone: +49 9191 180 siemens-healthineers.com

