Dimension® clinical chemistry system

Flex® reagent cartridge

LI

See shaded sections: Updated information from 2015-02 version.

Issue Date 2019-04-01

Lithium

Intended Use: The LI method used on the Dimension® clinical chemistry system is an *in vitro* diagnostic test intended to quantitatively measure lithium in human serum and plasma (sodium heparin). Measurements of lithium are used to assure that the proper drug dosage is administered in the treatment of patients with mental disturbances, such as manic-depressive illness (bipolar disorder).

Summary: Lithium, the third element in the periodic table, is used primarily to treat the manic phase of affective disorders, mania, and manic-depressive illness. The precise mechanism of action of lithium as a mood-stabilizing agent is not known. Lithium is administered in capsule, syrup, or tablet form as salts of either carbonate or citrate. It is readily absorbed from the gastrointestinal tract and does not bind appreciably to plasma proteins. Peak plasma concentrations are reached 2 to 4 hours after oral administration. Approximately 95% of a single dose of lithium is excreted in the urine within 6 to 12 hours, with the remainder being slowly excreted over the next 10 to 14 days.

Lithium concentrations are monitored to ensure patient compliance and prevent toxicity. Because there is a narrow therapeutic range of about 0.60 to 1.20 mmol/L, with significant risk of toxicity occurring above 1.5 mmol/L, determination of lithium concentration is crucial in the management of patients on lithium therapy. Since plasma values vary relative to time of last dose, a standardized 12-hour post-dose serum lithium concentration has been recommended to assess adequate therapy.^{1,2}

Principles of Procedure: The LI method employs a patented compound, 7-Nitro-2,12-dicarboxyl-16,17-dihydro-5H,15H-dibenzo[b,i] [1, 11, 4, 5, 7, 8]dioxatetraazacyclotradecine* (lithium dye) that reacts with Li⁺ ion in an alkaline mixture of water and dimethylsulfoxide to form a non-covalent binary complex.

The change in absorbance at 540 nm due to the formation of this non-covalent binary complex is directly proportional to the lithium concentration in the sample and is measured using a bichromatic (540 and 700 nm) endpoint technique.

*U.S. Patent #5,049,666

Reagents

Wells ^a	Form	Ingredient	Concentration ^b
1, 2	Liquid	Lithium Dye Reagent	
		Dye	1.53 x 10 ⁻⁴ M
		Dimethylsulfoxide (DMSO)	38% v/v
		K ₂ HPO ₄	0.65 mM
		Preservative	
3, 4	Liquid	Lithium Alkaline Salt Reagent	
		NaCl	8 mM
		Dimethylsulfoxide (DMSO)	38% v/v
		KOH	0.074 M
5, 6	Liquid	Lithium Blank Reagent	
		Dimethylsulfoxide (DMSO)	38% v/v
		K ₂ HPO ₄	0.65 mM
		Preservative	

a. Wells are numbered consecutively from the wide end of the cartridge.

b. Nominal value in final reaction mixture.

Risk and Safety:

H290, H319, H315, H317
P280, P305 + P351 + P338, P501

Warning!

May be corrosive to metals. Causes serious eye irritation. Causes skin irritation. May cause an allergic skin reaction.

Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents and container in accordance with all local, regional, and national regulations.

Contains: Potassium hydroxide; 5-chloro-2-methyl-3(2h)-isothiazolone mixture w/ 2-methyl-3(2h)-isothiazolone

Safety data sheets (MSDS/SDS) available on www.siemens.com/diagnostics

Precautions: Used cuvettes contain human body fluids; handle with appropriate care to avoid skin contact or ingestion.

For *in vitro* diagnostic use

Reagent Preparation: All reagents are liquid and ready to use.

Store at: 2 – 8 °C

Expiration: Refer to carton for expiration date of individual unopened reagent cartridges. Sealed cartridge wells on the instrument are stable for 30 days.

Open Well Stability: 5 days for wells 1 – 6

Specimen Collection and Handling: Normal procedures for collecting serum and plasma may be used for samples to be analyzed by this method.²

Follow the instructions provided with your specimen collection device for use and processing.³

Complete clot formation should take place before centrifugation.⁴

Serum or plasma should be separated from the cells if analysis is not performed within 2 hours of collection.² Once separated, lithium is stable for at least 24 hours at room temperature, 7 days if refrigerated at 4 °C.^{5,6} For longer storage, samples may be frozen at -18 °C for at least 6 months.⁵

Sodium heparin, present in blood collection tubes, does not interfere with the LI method.

Do not use blood collection tubes containing lithium heparin.

Procedure

LI Flex® reagent cartridge, Cat. No. DF132

Drug Calibrator, Cat. No. DC22B
Quality Control Materials

Materials Provided

Materials Required But Not Provided

Test Steps

Sampling, reagent delivery, mixing, processing, and printing of results are automatically performed by the Dimension® system. For details of this processing, refer to your Dimension® Operator's Guide.

Test Conditions

	Cuvette 1	Cuvette 2
Sample Size	10 µL	10 µL
Lithium Dye Reagent	0 µL	240 µL
Lithium Alkaline Salt Reagent	100 µL	100 µL
Lithium Blank Reagent	240 µL	0 µL
Temperature	37 °C	
Wavelengths	540 and 700 nm	
Type of Measurement	Bichromatic endpoint	

Calibration

Assay Range 0.20 – 5.00 mmol/L
Calibration Material Drug Calibrator, Cat. No. DC22B
Calibration Scheme 5 Levels, n = 2
Units mmol/L
Typical Calibration Levels 0.10, 0.90, 1.80, 3.60, 5.50 mmol/L
Calibration Frequency Every 60 days for any one lot.
A new calibration is required

- For each new lot of Flex® reagent cartridges
- After major maintenance or service, if indicated by quality control results
- As indicated in laboratory quality control procedures
- When required by government regulations

Assigned Coefficients
C₀ -185.0
C₁ 560.0
C₂ -0.5
C₃ 1.5
C₄ 0.5

Quality Control

At least once each day of use, analyze two levels of a quality control material with known lithium concentrations. Follow your laboratory internal QC procedures if the results obtained are outside acceptable limits.

Results: The instrument automatically calculates and prints the concentration of lithium in mmol/L using the calculation scheme illustrated in your Dimension® Operator's Guide.

Results of this test should always be interpreted in conjunction with the patient's medical history, clinical presentation and other findings.

Analytical Measurement Range (AMR): 0.20 – 5.00 mmol/L

This is the range of analyte values that can be directly measured on the specimen without any dilution or pretreatment that is not part of the usual analytical process and is equivalent to the assay range.

Samples with results in excess of 5.00 mmol/L should be repeated on dilution.

Manual Dilution: Make appropriate dilution with lithium-free serum to obtain results within the assay range. Enter dilution factor. Reassay. Resulting readout is corrected for dilution. Do not use water for manual dilutions.

Autodilution: Not recommended for this method.

Results less than 0.20 mmol/L should be reported as "less than 0.20 mmol/L" instead of the numerical value.

Limitations of Procedure

The instrument reporting system contains error messages to warn the user of specific malfunctions. Lithium results followed by such error messages should be held for follow-up. Refer to your Dimension® Operator's Guide.

A system malfunction may exist if the following 5-test precision is observed:

Concentration	SD
0.90 mmol/L	> 0.06 mmol/L
1.80 mmol/L	> 0.08 mmol/L

Interfering Substances

An increase in serum sodium concentration of 23 mmol/L from a nominal serum concentration of 140 mmol/L will produce biases of +0.06 and +0.05 mmol/L at lithium concentrations of 0.70 and 2.10 mmol/L, respectively.

A decrease in serum sodium concentration of 23 mmol/L from a nominal serum concentration of 140 mmol/L will produce biases of -0.06 and -0.05 mmol/L at lithium concentrations of 0.70 and 2.10 mmol/L, respectively.

Therapeutic Range: Therapeutic lithium concentrations vary significantly, depending on the individual.

A therapeutic range of 0.60 to 1.20 mmol/L includes effective serum/plasma concentrations for many patients; however, some individuals are best treated at concentrations outside this range.^{2,3,7}

Concentrations of 1.20 to 1.50 mmol/L signify a warning range, and a concentration in excess of 1.50 mmol/L in a specimen drawn 12 hours after the dose indicates a significant risk of intoxication.²

Concentrations greater than 2.00 mmol/L are often associated with toxic symptoms.^{2,3,7}

Note: The physician must ultimately determine the most appropriate lithium therapeutic range for each patient.

Specific Performance Characteristics⁴

Material	Precision ^{4,e}		
	Mean mmol/L	Standard Deviation (% CV)	
		Within-run	Total
MAS™ chemTRAK® Plus Control			
Level 1	0.52	0.02 (4.3)	0.02 (4.3)
Level 2	1.20	0.01 (0.9)	0.02 (1.3)
Level 3	1.92	0.02 (1.1)	0.02 (1.1)
Serum Pool 1	0.68	0.01 (2.1)	0.02 (3.4)
Serum Pool 2	1.83	0.02 (1.0)	0.02 (1.3)

c. All Specific Performance Characteristics testing was processed after normal recommended equipment quality control checks were performed (refer to your Dimension® Operator's Guide).

d. Reproducibility testing was done in accordance with the CLSI/NCCLS Approved Guideline for Evaluation of Precision Performance of Clinical Chemistry Devices (EP5-A, Feb. 1999).

e. Specimens at each level were analyzed in duplicate, once a day, for 20 days. The within-run and total standard deviations were calculated by the analysis of variance method.

MAS chemTRAK® is a registered trademark of Medical Analysis Systems, Camarillo, CA 93012.

Method Comparison

Comparative Method	Regression Statistics ^d			
	Slope	Intercept mmol/L	Correlation Coefficient	n
Ciba-Corning® ISE	1.01	-0.02	0.998	128 ^e

f. Model equation for regression statistics is: [Result of Dimension® system] = Slope x [result of comparative method] + Intercept.

g. The range of lithium values in the correlation study was 0.24 to 3.60 mmol/L.

Ciba-Corning® is a registered trademark of Ciba Corning Diagnostics Corp., Irvine CA, 92714.

Specificity

HIL Testing

The LI method was evaluated for interference from hemolysis, icterus and lipemia according to CLSI/NCCLS EP7-P. Bias, defined as the difference between the control sample (does not contain interferent) and the test sample (contains the interferent), is shown in the table below. Bias exceeding 10% is considered "interference".

Substance Tested	Test Concentration SI Units ^h	Lithium Concentration mmol/L	Bias ⁱ %
Hemoglobin (hemolysate)	1000 mg/dL	1.76	<10
Bilirubin (unconjugated)	[0.62 mmol/L] (monomer)	1.87	<10
Lipemia (Intralipid®)	[1368 µmol/L] [3000 mg/dL] [33.9 mmol/L]	1.66	<10

h. Système International d'Unités [SI Units] are in brackets.

i. Do not attempt to correct analyte concentration based on these results.

Intralipid® is a registered trademark of Fresenius Kabi AG, Bad Homburg, Germany.

Non-Interfering Substances

Interference testing was performed according to the CLSI/NCCLS Protocol EP7-P. The following substances do not interfere with the LI method when present in serum and plasma at the concentrations indicated. Systemic inaccuracies (bias) due to these substances are less than 10% at a lithium concentration of 0.62 to 1.79 mmol/L.

Substance	Test Concentration	SI Units
Acetaminophen	20 mg/dL	1323 µmol/L
Amikacin	15 mg/dL	256 µmol/L
Ampicillin	5 mg/dL	143 µmol/L
Ascorbic Acid	3 mg/dL	170 µmol/L
Caffeine	10 mg/dL	515 µmol/L
Calcium	166 mg/dL	15 mmol/L
Carbamazepine	12 mg/dL	508 µmol/L
Chloramphenicol	5 mg/dL	774 µmol/L
Chlordiazepoxide	2 mg/dL	67 µmol/L
Chlorpromazine	5 mg/dL	157 µmol/L
Cimetidine	10 mg/dL	396 µmol/L
Creatinine	30 mg/dL	2652 µmol/L
Dextran 75	2500 mg/dL	333 µmol/L
Diazepam	2 mg/dL	70 µmol/L
Digoxin	5 ng/mL	4 nmol/L
Erythromycin	20 mg/dL	273 µmol/L
Ethanol	350 mg/dL	76 mmol/L
Ethosuximide	30 mg/dL	2125 µmol/L
Furosemide	2 mg/dL	61 µmol/L
Gentamicin	12 mg/dL	251 µmol/L
Ibuprofen	40 mg/dL	1939 µmol/L
Lidocaine	6 mg/dL	256 µmol/L
Methylparaben	150 mg/dL	10 mmol/L
N-acetylcysteine	180 mg/dL	11 mmol/L
N-acetylprocainamide	2 mg/dL	72 µmol/L
Nicotine	2 mg/dL	123 µmol/L
Penicillin G	5 U/mL	25000 U/L
Pentobarbital	10 mg/dL	442 µmol/L
Phenobarbital	15 mg/dL	646 µmol/L
Phenytoin	10 mg/dL	396 µmol/L
Potassium	111 mg/dL	15 mmol/L
Primidone	10 mg/dL	458 µmol/L
Procainamide	180 mg/dL	7648 µmol/L
Propoxyphene	0.4 mg/dL	12 µmol/L
Protein, human albumin	6 g/dL	60 g/L
Protein, human IgG	6 g/dL	60 g/L
Quinidine	1 mg/dL	31 µmol/L
Rheumatoid Factor	500 IU/mL	500 IU/mL
Salicylic Acid	50 mg/dL	3620 µmol/L
Sodium Heparin	8 U/mL	8000 U/L
Theophylline	25 mg/dL	1388 µmol/L
Trimethylamine hydrochloride	100 mg/dL	10.5 mmol/L
Urea	500 mg/dL	83.3 mmol/L
Uric Acid	20 mg/dL	1.2 mmol/L
Valproic Acid	50 mg/dL	3467 µmol/L

Analytical Sensitivity: 0.04 mmol/L

The analytical sensitivity represents the lowest concentration of lithium that can be distinguished from zero. The sensitivity is defined as the mean value (n = 20) plus two standard deviations of a lithium free sample.

Symbols Key: See Adjacent Panel

Dimension® and Flex® are trademarks of Siemens Healthcare Diagnostics.

©2008 Siemens Healthcare Diagnostics
All rights reserved.

Dimension® clinical chemistry system

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©2008 Siemens Healthcare Diagnostics
Reservados todos los derechos.

Bibliography/Literatur/Bibliographie/Bibliografia/Bibliografía

1. Hardman JG, Goodman Gilman A, Limbird LE. Goodman and Gilman's The Pharmacological Basis of Therapeutics, Ninth Edition, The McGraw-Hill Companies, Inc., New York, NY 1996; pp. 446-453.
2. Burtis CA, Ashwood ER. Tietz Fundamentals of Clinical Chemistry, Fifth Edition, W.B. Saunders Company, Philadelphia, PA 2001; pp. 30-53 (specimen collection), pp. 631 (clinical significance), pp. 1023 (reference values).
3. Clinical and Laboratory Standards Institute/NCCLS. *Tubes and Additives for Venous Blood Specimen Collection; Approved Standard – Fifth Edition*. CLSI/NCCLS document H1-A5 [ISBN 1-56238-519-4]. CLSI, 940 West Valley Road, Suite 1400, Wayne, PA 19087-1898 USA, 2003.
4. Clinical and Laboratory Standards Institute/NCCLS. *Procedures for the Handling and Processing of Blood Specimens; Approved Guideline – Third Edition*. CLSI/NCCLS document H18-A3 [ISBN 1-56238-555-0]. CLSI, 940 West Valley Road, Suite 1400, Wayne, PA 19087-1898 USA, 2004.
5. Kaplan LA, Pesce AJ. Clinical Chemistry: Theory, analysis, and correlation, Third Edition, Mosby, Inc., St. Louis, MO 1996; pp. 1108 (specimen storage and reference values).
6. Young DS. Effects of Preanalytical Variables on Clinical Laboratory Tests, Second Edition, AACC Press, Washington, DC 1997; pp. 4-500 (specimen storage).
7. Baselt RC, Cravey RH. Disposition of Toxic Drugs and Chemicals in Man, Fourth Edition, Chemical Toxicology Institute, Foster City, CA 1995; pp. 430 (reference values).

Symbols Key Symbolschlüssel Explication des Symboles Interpretazione simboli Clave de los Símbolos

Do not reuse / Nicht zur Wiederverwendung / Ne pas réutiliser / Non riutilizzare / No reutilizar

Use By / Verwendbar bis / Utiliser jusque / Utilizzare entro / Fecha de caducidad

Batch Code / Chargenbezeichnung / Code du lot / Codice del lotto / Código de lote

Catalogue Number / Bestellnummer / Référence du catalogue / Numero di catalogo / Número de catálogo

Caution, consult accompanying documents / Achtung, Begleiddokumente beachten / Attention voir notice d'instructions / Attenzione, vedere le istruzioni per l'uso / Atención, ver instrucciones de uso

Manufacturer / Hersteller / Fabricant / Fabbicante / Fabricante

Authorized Representative in the European Community / Bevollmächtigter in der Europäischen Gemeinschaft / Mandataire dans la Communauté européenne / Mandatario nella Comunità Europea / Representante autorizado en la Comunidad Europea

Contains sufficient for <n> tests / Inhalt ausreichend für <n> Tests / Contenu suffisant pour "n" tests / Contenido sufficiente per "n" saggi / Contenido suficiente para <n> ensayos

In Vitro Diagnostic Medical Device / In-Vitro-Diagnostikum / Dispositif médical de diagnostic in vitro / Dispositivo medico-diagnostico in vitro / Producto sanitario para diagnóstico in vitro

Temperature Limitation / Temperaturbegrenzung / Limites de température / Limiti di temperatura / Limite de temperatura

Consult Instructions for Use / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultare le istruzioni per l'uso / Consulte las instrucciones de uso

Non-sterile / Nicht steril / Non stérile / Non sterile / No estéril

CE Mark / CE Zeichen / Marquage CE / Marchio CE / Marca CE

Contents / Inhalt / Contenu / Contenuto / Contenido

Reconstitution Volume / Rekonstitutionsvolumen / Volume de reconstitution / Volume di ricostituzione / Volumen de reconstitución

Level / Konzentration / Niveau / Livello / Nivel

2014-10_EFR6S

 Siemens Healthcare Diagnostics Inc.
500 GBC Drive
Newark, DE 19714 USA

Global Siemens
Headquarters
Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

Global Siemens
Healthcare Headquarters
Siemens AG
Healthcare Sector
Henkestrasse 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Global Division
Siemens Healthcare
Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591
USA
siemens.com/healthcare