

pH Validity Calibrators

See shaded sections:

Updated information from 2018-10 version.



10869627_US_D

pH Calibrators

1 Intended Use

The Syva® pH Validity Calibrators are used in the calibration of the Syva® pH Validity Test. These products are for forensic/toxicology use only.

2 Reagents

REF	Product Description	Volume
3T389UL	Syva® pH Validity Calibrator 2.0 synthetic urine with preservative at pH 2.0	14 mL
3T399UL	Syva® pH Validity Calibrator 3.0 synthetic urine with preservative at pH 3.0	14 mL
10736632	Syva® pH Validity Calibrator 4.0 synthetic urine with preservative at pH 4.0	14 mL
3T499UL	Syva® pH Validity Calibrator 4.5 synthetic urine with preservative at pH 4.5	14 mL
3T489UL	Syva® pH Validity Calibrator 9.0 synthetic urine with preservative at pH 9.0	14 mL
3T459UL	Syva® pH Validity Calibrator 11.0 synthetic urine with preservative at pH 11.0	14 mL
3T479UL	Syva® pH Validity Calibrator 12.0 synthetic urine with preservative at pH 12.0	14 mL

Risk and Safety

Safety data sheets (MSDS/SDS) available on siemens.com/healthcare

For forensic/toxicology use only.

Precautions

The calibrators must not be pipetted by mouth and should be handled using good laboratory practice to avoid skin contact or ingestion.

Preparation and Storage

The calibrators are provided ready to use and may be used directly from the refrigerator. No reconstitution is necessary. Remove the cap and mark it to identify it with its original vial. Always store the calibrators refrigerated at 2–8°C (36–46°F) when not in use.

Stability

When stored refrigerated at 2–8°C, the unopened calibrators are stable until the expiration date printed on the vial label. When stored refrigerated at 2–8°C, the opened calibrators are stable for 90 days.

3 Procedure

For procedures, refer to the Syva® pH Validity Test instructions for use.

4 Limitations

The Syva® pH Validity Calibrators are designed solely for use according to the procedures published in the appropriate Syva® reagent instructions for use.

5 Symbols Key

	Do not reuse / Nicht zur Wiederverwendung / Ne pas réutiliser / Non riutilizzare / No reutilizar
	Use By / Verwendbar bis / Utiliser jusque / Utilizzare entro / Fecha de caducidad
	Batch Code / Chargenbezeichnung / Code du lot / Codice del lotto / Código de lote
	Catalogue Number / Bestellnummer / Référence du catalogue / Numero di catalogo / Número de catálogo
	Caution, consult accompanying documents / Achtung, Begleiddokumente beachten / Attention voir notice d'instructions / Attenzione, vedere le istruzioni per l'uso / Atención, ver instrucciones de uso
	Manufacturer / Hersteller / Fabricant / Fabricante / Fabricante
	Authorized Representative in the European Community / Bevollmächtigter in der Europäischen Gemeinschaft / Mandataire dans la Communauté européenne / Mandatario nella Comunità Europea / Representante autorizado en la Comunidad Europea
	Contains sufficient for <n> tests / Inhalt ausreichend für <n> Tests / Contenu suffisant pour "n" tests / Contenuto sufficiente per "n" saggi / Contenido suficiente para <n> ensayo
	Temperature Limitation / Temperaturbegrenzung / Limites de température / Limiti di temperatura / Limite de temperatura
	Consult Instructions for Use / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultare le istruzioni per l'uso / Consultar las instrucciones de uso
	Non-sterile / Nicht steril / Non stérile / Non sterile / No estéril
	CE Mark / CE Zeichen / Marquage CE / Marchio CE / Marca CE
	Contents / Inhalt / Contenu / Contenuto / Contenido
	Reconstitution Volume / Rekonstitutionsvolumen / Volume de reconstitution / Volume di ricostituzione / Volumen de reconstitución
	Level / Konzentration / Niveau / Livello / Nivel
	For forensic/toxicology use only / Nur für den forensischen/toxikologischen Gebrauch / Pour une utilisation médico-légale ou toxicologique seulement / Ad esclusivo uso in medicina legale/tossicologica / Exclusivamente para uso forense o toxicológico

2015-03_EFIGS_FT

For technical assistance, call Siemens Healthcare Diagnostics:

1-800-227-8994 in the USA

1-800-264-0083 in Canada

Outside the USA and Canada, call your local Siemens representative.

and Syva® are trademarks of Siemens Healthcare Diagnostics.

© 2008 Siemens Healthcare Diagnostics
All rights reserved.

Siemens Healthcare Diagnostics Inc.
500 GBC Drive
Newark, DE 19714 USA

Global Siemens
Headquarters
Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

Global Siemens
Healthcare Headquarters
Siemens AG
Healthcare Sector
Henkestrasse 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens.com/healthcare

Global Division
Siemens Healthcare
Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591
USA
siemens.com/healthcare

Printed in USA
2019-08
10869627_US_D