



***Exclusion List for UniPOC
& Interface Servers***

Software Version 1.6

© 2011-2020 Siemens Healthcare Diagnostics. All rights reserved.

No part of this manual or the products it describes may be reproduced by any means or in any form without prior consent in writing from Siemens Healthcare Diagnostics.

UniPOC is a trademark of Siemens Healthcare Diagnostics.

All other trademarks are the property of their respective owners.

 Origin US
Siemens Healthcare Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591-5097 USA

**Global Siemens
Headquarters**
Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

**Siemens Healthineers
Headquarters**
Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Global Division
Siemens Healthcare
Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591-5097
USA
www.siemens.com/diagnostics

The information in this manual was correct at the time of printing.

However, Siemens Healthcare Diagnostics continues to improve products and reserves the right to change specifications, equipment, and maintenance procedures at any time without notice.

If the UniPOC system is used in a manner differently than specified by Siemens Healthcare Diagnostics, the protection provided by the equipment may be impaired. Observe warning and hazard statements.

Antivirus Exclusions

This document provides a list of the directory folders that should be excluded from daily scans or real time protection scans that are made by your antivirus software or any other scanning software.

Siemens recommends that an antivirus software package be installed with the exclusions listed below. There are no known incompatibilities with virus protection products and the UniPOC™ Data Management software.

Important: Make sure the directories specified below are set to “exclude subdirectories” as there are programs that don’t automatically exclude subdirectories.

Siemens recommends the OS be installed on the C: drive. On certain server configurations Siemens Healthineers related software is installed on the next logical drive available, usually the E: drive. Note, however, that some components are required to be installed on the C: drive.

“<DRIVE LETTER>” is to be replaced by the drive lettered partition wherever the installed program resides. Once the servers have been built, inspected and validated, Siemens can provide the proper partition letters.

“<VERSION>” is to be replaced by the installed version number of SQL Anywhere on the server.

“<INSERT BRAND HERE>” is to be replaced with the name of the antivirus software.

UniPOC – All-in-One Server Installation

IIS Specific

- C:\inetpub

MSMQ Specific

- C:\Windows\System32\msmq

SQL Server Specific

- <DRIVE LETTER>:\Program Files\Microsoft SQL Server
- <DRIVE LETTER>:\Program Files (x86)\Microsoft SQL Server

UniPOC Specific

- <DRIVE LETTER>:\Program Files (x86)\QCM3

Interface Specific

- C:\ProgramData\Sybase
- <DRIVE LETTER>:\Program Files (x86)\NNSY
- <DRIVE LETTER>:\Program Files (x86)\Sybase
- <DRIVE LETTER>:\Program Files (x86)\Conworx\Meditrac

Installation Engineer Specific

- <DRIVE LETTER>:\Software Installs

Anti-virus Client Specific

- <DRIVE LETTER>:\Program Files\<INSERT BRAND HERE>
- <DRIVE LETTER>:\Program Files (x86)\<INSERT BRAND HERE>

UniPOC Dedicated UI Server – Dual Server Installation

IIS Specific

- C:\inetpub

MSMQ Specific

- C:\Windows\System32\msmq

UniPOC Specific

- C:\Program Files (x86)\QCM3

Interface Specific

- C:\ProgramData\Sybase
- C:\Program Files (x86)\Sybase
- C:\Program Files (x86)\Conworx\Meditrac

Installation Engineer Specific

- C:\Software Installs

Anti-virus Client Specific

- C:\Program Files\<INSERT BRAND HERE>
- C:\Program Files (x86)\<INSERT BRAND HERE>

UniPOC Dedicated DB Server (Siemens Managed) – Dual Server Installation

SQL Server Specific

- <DRIVE LETTER>:\Program Files\Microsoft SQL Server
- <DRIVE LETTER>:\Program Files (x86)\Microsoft SQL Server

IIS Specific

- C:\inetpub

MSMQ Specific

- C:\Windows\System32\msmq

UniPOC Specific

- <DRIVE LETTER>:\Program Files (x86)\QCM3

Installation Engineer Specific

- <DRIVE LETTER>:\Software Installs

Anti-virus Client Specific

- <DRIVE LETTER>:\Program Files\<INSERT BRAND HERE>
- <DRIVE LETTER>:\Program Files (x86)\<INSERT BRAND HERE>

UniPOC Dedicated MeterCom Server – Triple Server Installation

IIS Specific

- C:\inetpub

MSMQ Specific

- C:\Windows\System32\msmq

UniPOC Specific

- C:\Program Files (x86)\QCM3

Installation Engineer Specific

- C:\Software Installs

Anti-virus Client Specific

- C:\Program Files\<INSERT BRAND HERE>
- C:\Program Files (x86)\<INSERT BRAND HERE>

Dedicated dNote Interface Server

IIS Specific -- (If i-STAT DE or WRX software is installed)

- C:\inetpub
- C:\Windows\System32\inetsrv
- C:\Windows\SysWOW64\inetsrv

SQL Server Specific (If dNote ADT or WRX software installed)

- C:\Program Files\Microsoft SQL Server
- C:\Program Files (x86)\Microsoft SQL Server

Interface Specific

- C:\ProgramData\Sybase
- C:\Program Files (x86)\INNSY
- C:\Program Files (x86)\Sybase

DE Software – (If i-STAT DE is installed on server)

- C:\IDEAUTOBACKUP
- C:\Program Files (x86)\APOC
- C:\Program Files (x86)\SQL Anywhere <VERSION>

Installation Engineer Specific

- C:\Software Installs

Anti-virus Client Specific

- C:\Program Files\<INSERT BRAND HERE>
- C:\Program Files (x86)\<INSERT BRAND HERE>