

Cybersecurity Bulletin

Date: 2021-07-09

Affected Products:

	Analyzer	Minimum Version	Operating System	Additional Requirements	Approx. Inst. Time (min)
1.	BCS XP	V1.1.1 for SBA 2.0	Windows 10 1809®	n/a	50-120 min
2.	Atellica COAG 360	V1.7	Windows 10 1809®	n/a	50-120 min
3.	Atellica NEPH 630	V1.1 for SBA 2.0	Windows 10 1809®	n/a	50-120 min
4.	BN ProSpec	V1.4.1 for SBA 2.0	Windows 10 1809®	n/a	50-120 min

Urgency:

☒ Critical

☐ Normal

Remark:

High Security Risk: If urgency is critical, immediate installation is recommended for affected product!

Normal Security Risk: If urgency is normal, installation is recommended for affected product!

Contact:

For further assistance, contact your local Siemens Healthcare Diagnostics representative.

Content

#	Manufacturer				
	Name	Id#, Version	Release Date	Description	Description Link
1.	McAfee	8.3.0-303	2020-04-30	McAfee Application Control and McAfee Change Control 8.3.0 -303	https://kc.mcafee.com/corporate/index?page=content&id=KB87944
2.	Microsoft	KB5004947	2021-07-06	Microsoft 06 July 2021 - Out-of-band update	https://support.microsoft.com/en-us/topic/july-6-2021-kb5004947-os-build-17763-2029-out-of-band-71994811-ff08-4abe-8986-8bd3a4201c5d
3.	Microsoft	KB5003255	2021-06-08	Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 1809	https://support.microsoft.com/en-us/topic/june-8-2021-kb5003255-cumulative-update-for-net-framework-3-5-and-4-8-for-windows-10-version-1809-and-windows-server-version-2019-df75fb75-142a-4d1a-8950-15f5f884f80e
4.	Microsoft	1.343.664.0	2021-07-09	Security intelligence updates for Windows Defender Antivirus and other Microsoft antimalware	https://www.microsoft.com/en-us/wdsi/defenderupdates
5.	Siemens	2.1.8	---	I2i VNC Server update	---
6.	Siemens	---	2021-04-06	I2i Agent Security Provider update	---
7.	Siemens	---	2021-04-22	I2i Agent Configuration modification for ANEPH 630	---
8.	Microsoft	KB4577586	2020-10-27	Microsoft removal of Adobe Flash	https://support.microsoft.com/en-us/help/4577586/update-for-removal-of-adobe-flash-player
9.	Microsoft	KB5003711	2021-06-08	Servicing stack update for Windows 10, version 1809	https://support.microsoft.com/en-us/topic/kb5003711-servicing-stack-update-for-windows-10-version-1809-june-8-2021-cc836592-f55c-4da9-820b-e4d3c5c5c1d1
10.	Microsoft	KB5000685	2021-02-24	Cumulative Update 23 for SQL Server 2017: Product version: 14.0.3381.3	https://support.microsoft.com/en-us/topic/kb5000685-cumulative-update-23-for-sql-server-2017-22b653c7-8487-4564-9db2-b5c1bd465145

Siemens Healthineers confirms that the patches included in this package, are solely intended to improve the cybersecurity of the system(s) and have no influence on the safe operation of the Siemens Healthineers products.

Please note:

This Cybersecurity Patch replaces all predecessor Cybersecurity Patches for Windows 10. Installing this latest released Cybersecurity Patch is equivalent to the installation of all predecessor Cybersecurity Patches.

The patches included in this package can only be installed if all preconditions are fulfilled. This may include:

- Proper operating system version
- Proper service pack installed
- Precursor patches are installed

Installation Instructions

Installation via Secure Download:

Doc Lib: <https://www.healthcare.siemens.com/doclib>

- Use the URL for download of the released security patches.
- Copy the downloaded file to a mobile storage device (USB Flash Drive)
- Transfer the file to the instrument computer

Note: One or more restarts of the computer, which require(s) password entry, may be needed.

For further information refer to

BCS ® XP System Secure Download Software Instruction Manual
BN ProSpec ® System Secure Download Software Instruction Manual
Atellica COAG 360 System Instruction Manual
Atellica NEPH 630 System Instruction Manual