

## RxOnly




This guide provides first time setup instructions for the epoc® Reader. For complete user instructions, please refer to the epoc System Manual.

## Package Contents

- epoc Reader
- AC Adapter power supply
- Sheet of labels
- Sheet of clear label overlays

## epoc Reader Components

The figure below shows the Reader's primary components and status light indicators.

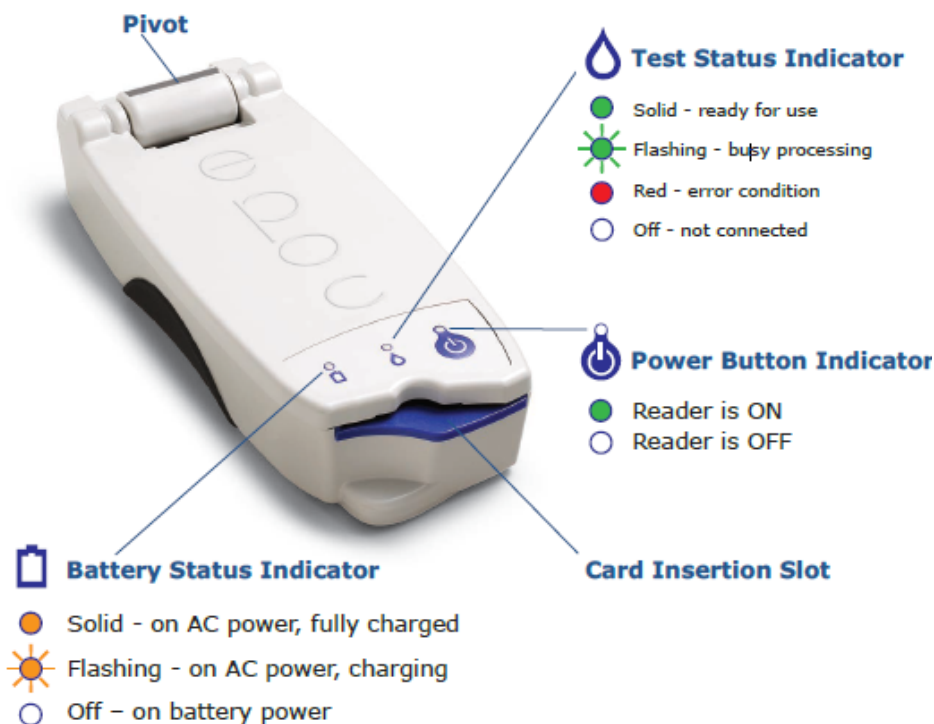

***Note:** Check Document Library for any applicable updates prior to using this device.*


## Step 1: Charge

Before using the epoc Reader, charge its battery as follows:

1. Make sure the Reader is OFF. If the Power Button Indicator is green, press and hold the power button for at least three seconds to turn the Reader off.
2. Plug the AC Adapter into a wall socket.

3. Plug the AC Adapter into the power jack on the back of the Reader.
  - The amber battery status indicator will start blinking
  - When the battery has been fully charged, the amber battery status indicator will stop blinking

**Note:** Initial charge will take approximately four hours.

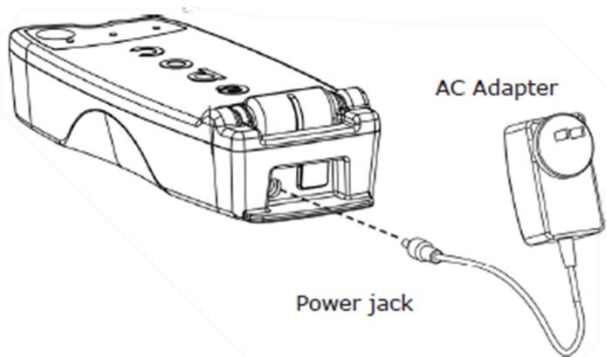

## Step 2: Label

Label the epoc Reader to help easily identify it from other Readers in your facility. Use the included labels and place the clear overlay over the label to increase its durability.

**Note:** Before placing this unit into service, consult your facility's internal quality control procedures and regulatory requirements.

## Care and Maintenance

Refer to Section 10 of your epoc System Manual for care and maintenance information, including recommended cleaning methods.

**Note:** If you are uncertain about the compatibility of a specific cleaning or decontamination agent, contact your service representative.

## Revision History

Rev. 11	Updated header and revision; added section, "Care and Maintenance"
Rev. 10	Updated contact information and revision, replaced symbols, minor administrative changes



**Epocal Inc.**  
2060 Walkley Rd.  
Ottawa, ON K1G 3P5 Canada  
[siemens-healthineers.com/epoc](http://siemens-healthineers.com/epoc)



Chapel Lane  
Swords, Co. Dublin, Ireland  
[siemens-healthineers.com/poc](http://siemens-healthineers.com/poc)

**Siemens Healthcare Diagnostics  
Manufacturing Ltd.**



© 2024 Siemens Healthcare Diagnostics Inc. All rights reserved.

epoc and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates.

51000848-EN Rev.: 11 2024-04