



dNote System Requirements

Software Version 3.3, Patch 2

© 2011-2024 Siemens Healthcare Diagnostics, Inc. All rights reserved.

No part of this manual or the products it describes may be reproduced by any means or in any form without prior consent in writing from Siemens Healthcare Diagnostics Inc.

All other trademarks are the property of their respective owners.

 Origin DE
Siemens Healthcare Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591-5097 USA

**Global Siemens
Headquarters**
Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

**Siemens Healthineers
Headquarters**
Siemens Healthcare AG
Siemensstr. 3
91301 Forchheim
Germany
Phone: +49 9191 18-0
siemens-healthineers.com

Global Division
Siemens Healthcare
Diagnostics Inc.
511 Benedict Avenue
Tarrytown, NY 10591-5097
USA
www.siemens.com/diagnostics

The information in this manual was correct at the time of printing. However, Siemens Healthcare Diagnostics Inc., continues to improve products and reserves the right to change specifications, equipment, and maintenance procedures at any time without notice.

If the dNote system is used in a manner differently than specified by Siemens Healthcare Diagnostics Inc., the protection provided by the equipment may be impaired. Observe warning and hazard statements.

Server Requirements for dNote Version 3.3

System Software

Web Engine:	Microsoft Internet Information Server (IIS) 10.0
Database:	Microsoft SQL Server 2019 (64-bit) Express (Included with the dNote product)
Virus Protection:	Any antivirus solution can be used. Exclusions from real-time scans must be configured for all installations <u>after</u> all application software is installed. (See <i>Exclusion List for UniPOC & Interface Servers</i> for details.)
Other Software:	Microsoft .NET Framework 3.5 SP1 Microsoft .NET Framework 4.8 Microsoft OLE DB Driver 18 for SQL Server Microsoft Message Queuing (MSMQ) v3.0 (for the UniPOC UI/dNote DataServer Service Server) Supports all versions of UniPOC on Windows Server 2016 and 2019

System Specifications

Dedicated dNote / WRX Server Requirements (Minimum)

Processor	RAM	HD Capacity	Operating System
4 core processor	16 GB or greater	200 GB or greater. RAID 5 or RAID 10 configuration strongly recommended. IMPORTANT: Fully allocated to single "C" drive.	Windows Server 2016 Version 1607 (OS Build 14393.2906) Windows Server 2019
Remote Access	Network	Backups	Additional Software
Broadband connection for remote maintenance with Siemens Smart Remote Services (SRS) application.	100baseT NIC or greater	Network Backup Solution	Antivirus software and disk defragmentation tools are required. Microsoft Edge (Chromium) Siemens Healthineers recommends you use HTTPS protocol. Note: Customers are responsible for providing the SSL certificate.

Client Requirements for dNote / WRX

Client Software

Web Browser	Microsoft Edge Chromium JavaScript enabled Other (or older) browsers may function but are not supported
--------------------	---